


Q-LAS PCL5


Overview

A.R.C. LASER

Control Panel Interface

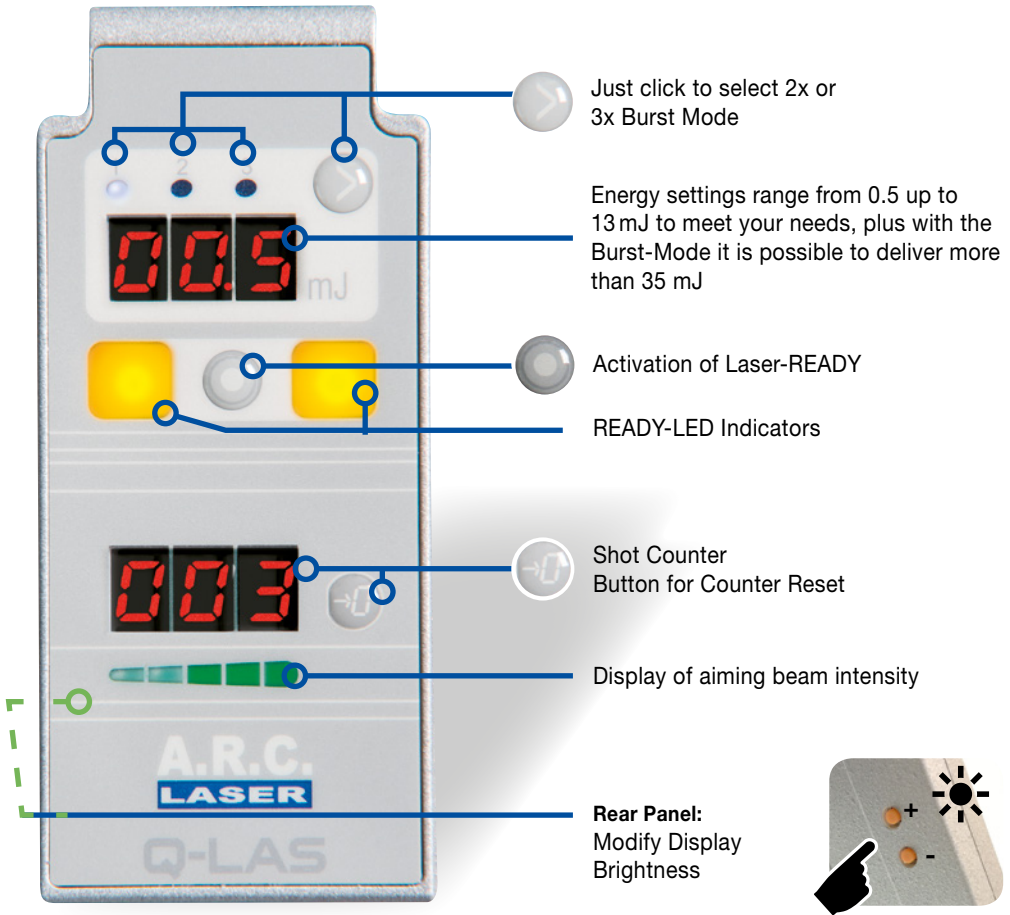

Power Control

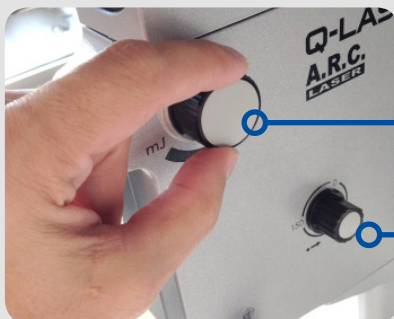

Adjustment of output energy is made directly on the laser module.

Focus Shift,
Set the plain of laser focus

Q-LAS PCL5

After you switched the Laser on.

A.R.C.
LASER

1) The Warm-up

During this process the Q-Las counts to ZERO



When finished, the system remembers the last Power-value Setup.

Here the Q-LAS now expects your manual calibration.

2) Calibration

Set the laser to a power value of **05.0 mJ**



symbolic picture



Set the power to 05.0 mJ



The calibration starts as soon as you set the value of 05.0 mJ.

Please NOTE: An ERROR occurs when you continue turning the dial.

3) Calibration Process

Now the system automatically runs the calibration process - the READY-LEDs are indicating that with several blinks.



Ready to set your desired treatment parameters ...



2-seconds

4) Alter power level and start treatment

Please keep in mind that 05.0 mJ is a high level of power. You have to change the parameter. Otherwise the laser will not start.

After changing the parameter and pressing the READY button the system will blink for 2 seconds, then you can start your treatment.